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DEVICE AND ACCESSORIES / CONSUMABLES DESCRIPTION

JUVAPEN Expert is a cordless motorized device intended to push a syringe plunger in a controlled manner, in order to assist the practitioner in injecting Botulinum toxins for aesthetic treatments.

JUVAPEN Expert is used with the following accessories and consumables:

- One syringe-holder that links the **JUVAPEN Expert** to a compatible syringe. Its lever allows to activate the injection with a simple pressure on the top part of the lever close to the syringe extremity. (ref.: JE02 - Class I medical device)
- One 6V JUVAPLUS Lithium battery (ref.: BP04)
- One protective sleeve (ref.: PS01 - Class I medical device)
- One 1ml breakable plunger rod syringe identical to those offered by JUVAPLUS (ref.: JP03 - Class Is medical device)
- One standard needle (not provided - Class Ila medical device)

The use of other accessory or consumable could compromise the proper functioning of **JUVAPEN Expert**.

ADVANTAGES OF THE JUVAPEN Expert

For the patient	For the practitioner
<ul style="list-style-type: none"> • Significantly less pain • Highly precise injection • Reduced side effects 	<ul style="list-style-type: none"> • Easy to use & intuitive system • Optimized control of the procedure: depth, homogeneity • Accuracy and flexibility in dosage

GENERAL RECOMMENDATIONS

JUVAPEN Expert is intended to be used by a medical practitioner specialized in botulinum toxin injections.

JUVAPEN Expert is intended to be used on any person indicated to receive botulinum toxin injections, independently of age or sex, depending on the judgement of the practitioner.

The choice of patients, injectable solution and injection mode are under his / her sole responsibility. Therefore the results are the exclusive responsibility of the practitioner who only can evaluate the indications and contraindications related to the technique to employ and the product to be administered.

The medical practitioner shall use **JUVAPEN Expert** only after having fully familiarized himself / herself with both the product and its use. The practitioner must ensure the **JUVAPEN Expert** is functioning properly before use.

USUAL CAUTIONS

- Do not use if the package is open or damaged.
- Keep the **JUVAPEN Expert** out of reach of children.
- Do not use in an inflammable environment.
- Do not press the buttons while inserting the battery.
- Any alteration of the **JUVAPEN Expert** may cause dysfunctions.
- WARNING : The use of **JUVAPEN Expert** next to or stacked with other electrical equipment must be avoided as it may cause dysfunction. Please refer to the EMC section (last page).
- Do not expose **JUVAPEN Expert** to the sunlight, and observe handling conditions.
- The battery used in this device may present risk of fire if damaged or badly replaced. Do not open, recharge, incinerate, dispose of in fire, heat above 100°C (212° F), short circuit, do not sterilise, put in backwards, do not mix with other battery. CAUTION : Risk of explosion if the battery is not replaced properly. Replace only by the same type or equivalent type.

CONTRAINDICATIONS

There are no contra-indications associated with the use of **JUVAPEN Expert**. However, possible side effects and contraindications can result from the use of products in conjunction with **JUVAPEN Expert** (Toxin, needle, ...).

It is therefore the practitioner's full responsibility to avoid any adverse reactions or complications associated with the use of these products.

If the device is damaged, contact the customer service (refer to the warranty).

CLEANING

JUVAPEN Expert and its syringe holder are not sterile. It is recommended to clean them after each use, with a lightly damp cloth without chemical products nor detergent.

- Do not immerse in liquids.
- Do not put in the washing machine.
- Do not autoclave

The practitioner ensures that **JUVAPEN Expert** and the syringe holder are cleaned according to the current local regulation.

HANDLING CONDITIONS

It is recommended to store and transport **JUVAPEN Expert** in its original packaging in the following conditions:

	OPERATING 0 - 2000 m	TRANSPORT & STORAGE
Temperature	20°C 68°F	25°C 77°F
Humidity	60% RH at 86°F / 30°C non condensing	

RECYCLING

- Throw the battery in conformity with the current environmental regulation.
- The syringe holder may be used several times before being thrown in the bin unless different local registration.
- The syringe, the protective sleeve and the needle are single-patient use and must be disposed of immediately after use in the appropriate bin. Do not use if their packaging is open or damaged.
- Do not throw the **JUVAPEN Expert** but return to JUVAPLUS SA.

WARRANTY

The warranty period of **JUVAPEN Expert** is 24 months.

The warranty period of the syringe holder is 3 months.

ASSEMBLY AND USE (refer to the Quick start guide included).

1. Fill the syringe with the desirable volume of toxin, and attach the needle to the syringe.
2. Prime the air in the syringe.
3. Break the plunger rod at the chosen volume of toxin.
4. Insert the device into its single use protective sleeve.
5. Attach the syringe holder to the **JUVAPEN Expert** by rotating it until you hear a click.
6. Insert the syringe into the syringe holder by uplifting the activation lever and sliding the syringe downwards until you hear a click. Then lower the activation lever and take the protection cap off the needle.
7. Turn the device on by pressing the **central button**. The green light and the screen showing **CONTINUOUS** mode confirms that **JUVAPEN Expert** is now operational.
- 8/9. Remove the air from the syringe by pressing the activation lever until the syringe is fully primed.
10. Select the **DOSE** mode by pressing the right button.
11. Highlight the product selection zone by pressing the down button.
12. Select the desired product by navigating with left and right buttons.

Only for *Custom product*

- 12.1 Press the **central button** to set the dilution.
- 12.2 Select the number of diluted units by navigating with left and right buttons.
- 12.3 Highlight the dilution volume selection zone by pressing the bottom button.
- 12.4 Select applied dilution volume navigating with left and right buttons.
- 12.5 Press the **central button** to validate the applied dilution and come back to the **DOSE** mode of the **JUVAPEN Expert**.
13. Highlight the **DOSE** selection area by pressing the bottom button.
14. Select the desired dose by navigating with left and right buttons. The **DOSE** is shown in Unit(s) and millilitres for all products, except for the product Volume which only displays millilitres.
15. Press the activation lever to inject the selected dose.
16. Once the syringe is empty replace the needle cap.
17. The **REWIND** mode displays automatically when the syringe is empty. If not, highlight the mode selection area by pressing once or twice (depending on your position in the **DOSE** mode) the top button. Then, press the right button to enter the **REWIND** mode.
18. Press the activation lever once to bring the screw back to its original position.
19. Once the screw is entirely retracted, take the syringe off by up lifting the lever and having it slide upwards. Dispose of the syringe with the needle.
20. Disconnect the syringe holder from the **JUVAPEN Expert** by rotating it.
21. Take the single use protective sleeve off and bin it.
22. The **JUVAPEN Expert** shuts off after 2 minutes when not in use. To switch off the **JUVAPEN Expert** manually, press the **central button** till the extinction of the light indicator and the screen.

USER INTERFACE


Status light indicator
Activation **button** – activate by pressing on the activation lever of the syringe holder.

Screen

Selection Button

Selection Button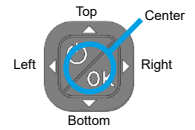

Light indicator	Indication	Recommended action
Continuous green	Ready for use	Normal use
Blinking green	Continuous mode activated	Normal use CONTINUOUS mode
Blue to green gradation	Ongoing dose injection	Normal use during injection
Slow blinking orange	Return mode activated	Normal use REWIND mode
Fast blinking orange	1/ Plunger at endpoint 2/ Clogged needle	1/ JUVAPEN Expert is in REWIND mode 2/ Change the needle

USE OF THE GRAPHIC INTERFACE

CONTINUOUS mode
(FWD)

DOSE Mode
(DOSE)


REWIND Mode
(RWD)

JUVAPEN Expert disposes of three distinct use modes : **CONTINUOUS**, **DOSE** et **REWIND**.

In **CONTINUOUS** and **REWIND** mode.

the user can act on two areas: mode selection area and sound activation/deactivation area. Two other information areas are displayed on the screen.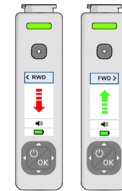


Mode selection area
Area indicating the screw motion
Area for sound activation/deactivation
Zone Area indicating the battery level


	Battery level	Action
	Normal	-
	Low	Foresee renewal of the battery
	Very low	Replace the battery

Sound activation/deactivation

JUVAPEN Expert can be set to emit a sound (bip) when one of the **selection button** is pressed. The change between sound activated/deactivated and vice versa can be done by pressing the **central button** after having highlighted the dedicated zone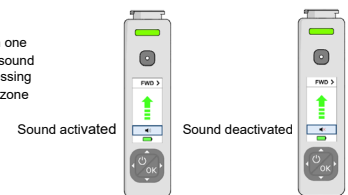

In **DOSE** mode

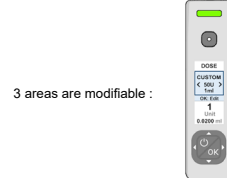3 areas are modifiable :

Mode selection area
Product selection area
Dose selection area

Refer to the attached Quick Start Guide for product and dose selection

When "*Custom*" product is selected and the *product* selection area is activated (highlighted) the text «Ok : Edit» is visible.

Custom product : Selection area for the *product* with text «Ok : Edit»

By pressing the **central button**, the *product* dilution selection submenu is displayed.

Number of unit selection area

Dilution volume selection area

During dose administration: While injecting, the screen displays the number of unit and the volume corresponding to the ongoing injection.

The injection progress is displayed with a bargraph.

Reminder area for number of unit and volume corresponding to the ongoing injection
Ongoing injection progression area

EMC Information

JUVAPEN Expert is classified Class B/Group 1 in emission. **JUVAPEN Expert** complies with the standard IEC 60601-1-2: 2014.

Medical electrical equipment should be used with precautions according to EMC notices disclosed in this manual.

JUVAPEN Expert is intended for use in the electromagnetic environment specified below. The customer or user of **JUVAPEN Expert** should assure that it is used in such an environment.

Table 1 – Emission Requirements Comparison		
Emission test	Compliance	Electromagnetic environment - guidance
Conducted emissions CISPR 11: 2009 / AMD1: 2010	Not applicable	<p>JUVAPEN Expert uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.</p> <p>JUVAPEN Expert is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.</p>
Radiated emissions CISPR 11: 2009 / AMD1: 2010	Class B / Group 1	
Harmonic emissions IEC 61000-3-2:2018	Not applicable	
Voltage fluctuations flicker emissions IEC 61000-3-3: 2013/AMD1:2017	Not applicable	
Devices with motors or switching circuits CISPR 14-1	Not applicable	

Table 2 – Immunity Level Comparison – Transient Phenomenon			
Immunity test	IEC60601 test level	Verdict	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2: 2008	+/- 8 kV contact discharge +/- 15 kV air discharge	Pass	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient / burst IEC 61000-4-4: 2012	+/- 2 kV AC Mains +/- 1 kV I/O Ports 100 kHz PRR	Not applicable	<p>Mains power quality should be that of a typical commercial or hospital environment.</p> <p>JUVAPEN Expert is a self-powered (non-rechargeable battery) handy device that is not connected to the public low-voltage power supply network.</p>
Surge – AC Mains IEC 61000-4-5: 2014 AMD1: 2017	+/- 2 kV (max) 0,90, 180 & 270 ° phase angles	Not applicable	
Surges – 12 VDC Power ISO 7637-2	600 V	Not applicable	
Voltage dips & short interruptions immunity IEC 61000-4-11: 2004 AMD1:2017	UT=0%, 0,5 cycle (0, 45, 90, 135, 180, 225, 270 & 315°) UT=0%; 1 cycle – UT=70% 25/30 cycles (0°) UT=0%; 250/300 cycles	Not applicable	
Conducted Immunity IEC 61000-4-6:2013	3V (0.15-80MHz) 6V (ISM bands)	Not Applicable	
Magnetic Immunity IEC 61000-4-8: 2009	30 A/m 50 and 60 Hz	Pass	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

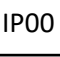
Table 4 – Immunity Level Comparison – Steady State Field Phenomenon		
Immunity test	IEC60601 test level	Verdict
Radiated RF IEC IEC 61000-4-3: 2006 AMD1:2007/AMD2:2010	10V/m 80 MHz to 2,7GHz	Pass
Proximity Field from Wireless Transmitters IEC 61000-4-3: 2006 AMD1:2007/AMD2:2010	9V/m to 28 V/m 15 specific frequencies	Pass
<p>Electromagnetic environment – guidance</p> <p>Portable and mobile RF communications equipment should be used no closer to any part of the JUVAPEN Expert, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended Separation Distance $d = 1.17 \sqrt{P}$ 0.15 MHz to 80 MHz - $d = 1.17 \sqrt{P}$ 80 MHz to 800 MHz - $d = 2.33 \sqrt{P}$ 800 MHz to 2,7GHz, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer, and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 		

Table 6 – Recommended separation distances between portable and mobile RF communications equipment and the JUVAPEN Expert			
JUVAPEN Expert is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the JUVAPEN Expert can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and JUVAPEN Expert – as recommended below, according to the maximum output power of the communications equipment.			
Rated Maximum Output Power of Transmitter (W)	Separation Distance According to Frequency of Transmitter (m)		
	150 kHz à 80 MHz $d = 1.17 \sqrt{P}$	80 MHz à 800 MHz $d = 1.17 \sqrt{P}$	800 MHz à 2,7GHz $d = 2.33 \sqrt{P}$
0.01	0.12 m	0.12 m	0.23 m
0.1	0.37 m	0.37 m	0.74 m
1	1.17 m	1.17 m	2.33 m
10	3.7 m	3.7 m	7.37 m
100	11.70 m	11.70 m	23.30 m

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